
Mölnlycke®

Mepitac®




WITH SAFETAC® TECHNOLOGY

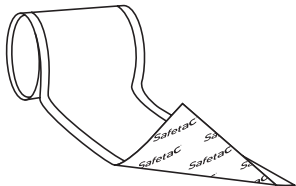Manufacturer/Fabricant

 **Mölnlycke Health Care AB**
Gamlestadvägen 3C, Box 13080
SE-402 52 Göteborg, Sweden

Mepitac®

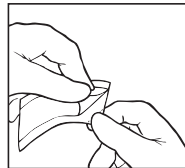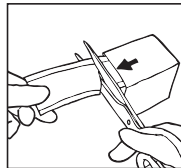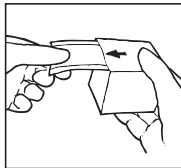Single use
Usage unique

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Toll free number :
USA 1-800-882-4582
Canada 1-800-494-5134

UK Responsible Person:

Mölnlycke Health Care Ltd.,
Medlock Street, Oldham, Lancashire, OL1 3HS, UK

Australian sponsor address:

Mölnlycke Health Care Pty. Ltd.
12 Narabang Way, Belrose, NSW 2085 AUSTRALIA

- en** Medical Device
- de** Medizinprodukt
- fr** Dispositif médical
- es** Producto sanitario
- nl** Medisch hulpmiddel
- sv** Medicinteknisk produkt
- it** Dispositivo medico
- fi** Lääkinnällinen laite
- pt** Dispositivo médico
- da** Medicinsk udstyr
- el** ιατροτεχνολογικό προϊόν
- pl** Wyrób medyczny
- cs** Zdravotnický prostředek
- hu** Orvostechnikai eszköz
- no** Medisinsk utstyr
- sl** Medicinski pripomoček
- bg** Медицинско изделие
- ro** Dispozitiv medical
- sk** Zdravotnícka pomôcka
- tr** Tıbbi cihaz
- lt** Medicinos priemonė
- lv** Medicīniska ierīce
- et** Meditsiiniseade
- ru** Медицинское изделие
- hr** Medicinski proizvod
- sr** Medicinsko sredstvo
- ar** جهاز طبي

Product description

Mepitac is a conformable adhesive tape which is easy to apply and remove. The product is suitable for patients requiring repeated application and removal as it offers gentle removal from skin and bodily hair and provides secure fixation, yet leaves minimal residues on removal. If inspection or repositioning is necessary, Mepitac can be lifted and re-applied. Mepitac is also water resistant.

Mepitac consists of:

- a Safetac® contact layer
- a conformable knitted fabric
- a breathable and waterproof polyurethane film

Material content:

Polyurethane film, acrylic adhesive, polyester/polyamide tricot, silicone adhesive, polyethylene film

Indications for use

Mepitac is intended for fixation of medical devices such as drains, tubes, probes, electrodes, IV cannula, and dressings. Mepitac offers gentle skin protection when used under devices such as tubes.

Warning

Mepitac is not intended for fixation of life-saving and life-sustaining devices.

Precautions

- The adhesion of Mepitac may decrease when used in humid condition such as incubators or on patients who have a fever or perspire a lot.
- Do not reuse. If reused performance of the product may deteriorate, cross contamination may occur.
- Do not use Mepitac on patient and/or user with known hypersensitivity to the ingoing materials/components of the product.

Instructions for use

Mepitac can be used by lay persons under supervision of health care professionals.

1. Make sure the skin is clean and dry and free from any ointment etc.
2. Cut off the desired length of tape using scissors.
3. Remove the protective release film and apply Mepitac without stretching.
4. If inspection or re-positioning is necessary, Mepitac can be lifted and re-applied. The change interval is dependent on skin condition, change Mepitac when adhesion properties are no longer sufficient.

Disposal should be handled according to local environmental procedures.

Other information

If any serious incident has occurred in relation to the use of Mepitac it should be reported to Mölnlycke Health Care and to your local competent authority.

Mepitac® and Safetac® are registered trademarks of Mölnlycke Health Care AB.

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