Mölnlycke®

# Mextra® Superabsorbent

**STERILE** **EO**

CE 2797

Manufactured in China for

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# Mextra® Superabsorbent

Superabsorbent dressing with a fluid-repellent backing

## Product description

Mextra Superabsorbent is a superabsorbent dressing with a fluid-repellent nonwoven backing, a core consisting of superabsorbent particles and a nonwoven wound contact layer. Mextra Superabsorbent absorbs wound exudate through the wound contact layer, retains the exudate within the core and reduce the risk for maceration. The fluid-repellent nonwoven backing acts as a barrier and prevents exudate strike-through.

Mextra Superabsorbent consists of:

- A nonwoven wound contact layer
- Absorbent pad layer with superabsorbent particles
- A breathable nonwoven backing material

Dressing material content:

Polyolefin, cellulose, polyester, polyacrylate

## Indications for use

Mextra Superabsorbent is intended for use on moderately to heavily exuding wounds.

## Precautions

- Do not use Mextra Superabsorbent on patient and/or user with known hypersensitivity to the materials/components of the product.
- If you see signs of infection e.g. fever or the wound or surrounding skin becoming red, warm or swollen, consult a health care professional for appropriate treatment.
- Sterile. Do not use if inner package (sterile barrier) is damaged or opened prior to use. Do not re-sterilise.
- Do not reuse. If reused performance of the product may deteriorate, cross contamination may occur.

- Do not use Mextra Superabsorbent in cavities.
- Do not cut the Mextra Superabsorbent dressing.

## Instructions for use

Mextra Superabsorbent can be used by lay persons under supervision of health care professionals.

Mextra Superabsorbent can be used where high absorbency is required as a primary dressing directly on the wound or as a secondary dressing.

Mextra Superabsorbent can be used in conjunction with compression therapy for venous leg ulcers.

1. Cleanse the wound in accordance to clinical practice.
2. Select a dressing size so that the wound pad overlaps the wound with at least 2cm. Do not cut the product.
3. Apply Mextra Superabsorbent directly on the wound area, or as a secondary dressing over a dressing that transfers exudate. The product shall be placed with the white side of the product facing the wound and the blue back side facing up.
4. Secure the dressing e.g. with a suitable bandage or tape.

Mextra Superabsorbent should be changed according to the clinical condition of the wound, or before saturated. If the condition of the wound deteriorates, discontinue use and consult a healthcare professional. Maximum wear time of each dressing is 7 days.

Disposal should be handled according to local environmental procedures.

## Other information

If any serious incident has occurred in relation to the use of Mextra Superabsorbent it should be reported to Mölnlycke Health Care and to your local competent authority.

Mextra® is registered trademark of Mölnlycke Health Care AB.

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